



**AirFit™ F20**



## Fitting guide for AirFit F20



- Start by ensuring the magnets on the lower straps are unclipped.
- If they are clipped, detach them by twisting and pulling them away from the frame.



- Hold the mask to the face, ensuring the top of the mask is aligned with the top of the nasal bridge.
- Pull the headgear over the head, making sure the logo is facing outward.



- Position the lower straps so they both sit below the ears.
- Bring each of the magnets up to meet their corresponding clips on the frame.



- Adjust the fastening tabs on the upper straps until the mask feels even and stable.
- Reattach the fastening tabs.



- Repeat step 4 with the lower straps.
- Reattach the fastening tabs.



- Connect the device's air tube to the elbow, then click the elbow into the front of the mask.
- Start the device.



### Additional options

For patients with longer hair, we recommend feeding their hair through the back of the headgear. If tied in a pony tail, it should sit high enough to fit comfortably through the back of the headgear.

Refer to the user guide for further instructions and complete labeling information including indications, contraindications, warnings and precautions.

1 ResMed AirFit F20 internal Australian fitting study of existing CPAP patients conducted between March–April 2016. 2 ResMed AirFit F20 internal USA fitting study of existing CPAP patients conducted April 2016. 3 ResMed AirFit F20 internal EU and APAC fitting study of existing CPAP patients conducted June 2016. 4 Combined responses of those expressing a preference in an independent patient survey in the UK, USA, France, Germany and Australia between April 10 to May 11, 2015. Visit ResMed.com/MaskBrand.

Distributed by ResMed Corp, 9001 Spectrum Center Boulevard, San Diego, CA 92123 USA +1 858 836 5000 or 1 800 424 0737 (toll free). See ResMed.com for other ResMed locations worldwide. AirFit and InfinitySeal are trademarks and/or registered trademarks of the ResMed family of companies. Specifications may change without notice. © 2016 ResMed Ltd. 1019453/2 2016-10