

Reorder Information

Part Number	Description	Mask Color
100332	SleepWeaver Advance Nasal Mask & Headgear	Beige
100336	SleepWeaver Advance Nasal Mask ONLY	Beige
100274	SleepWeaver Advance Nasal Mask & Headgear	Blue
100289	SleepWeaver Advance Nasal Mask ONLY	Blue
100277	SleepWeaver Advance Nasal Mask & Headgear	Pink
100291	SleepWeaver Advance Nasal Mask ONLY	Pink
100281	SleepWeaver Advance Nasal Mask & Headgear	Camo
100293	SleepWeaver Advance Nasal Mask ONLY	Camo
100285	SleepWeaver Advance Nasal Mask & Headgear	Leopard
100295	SleepWeaver Advance Nasal Mask ONLY	Leopard
100239	SleepWeaver Advance Headgear ONLY	Black

SleepWeaver Parts Included

SleepWeaver Mask and Swivel




Swivel

One Piece Headgear



CPAPXCHANGE

SLEEP WELL TONIGHT

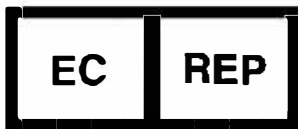

EMERGO EUROPE
Molenstraat 15
2513 BH, The Hague
The Netherlands
Tel: +31 (0)70 345 8570



0459

Circadiance

1060 Corporate Lane • Export, PA 15632
Phone: 888 • 825 • 9640 or +1 724 • 858 • 2837
cservice@circadiance.com
www.sleepweaver.com

Assembled in Mexico
with US made parts.

100341 REV5

SleepWeaver[®] ADVANCE

Cloth Nasal Cpap Mask and Headgear

LATEX FREE



Introduction to Your Mask

Your SleepWeaver Advance Mask is attached to headgear, as shown at left. The headgear has three adjustment points—one on each side of the mask and one on the Head Strap. A swivel connector attaches to the hose from your CPAP or bilevel machine.

Intended Use

The SleepWeaver is intended for both individual use and multiple patient re-use as a prescribed continuous airway pressure or bilevel therapy for use in the home and Sleep Labs. The SleepWeaver has been tested at pressures from 4 cm H₂O up to 20cm H₂O on CPAP and Bi-Level systems. Please watch the “Instructional Video” at www.circadiance.com/help.

Fitting the Mask

1. Remove your SleepWeaver Advance Mask from the packaging.
2. Connect the tubing from your CPAP or bilevel machine to the Swivel Connector on the front of the mask.
3. Turn on your CPAP or bilevel machine at your prescription pressure.
4. When the mask inflates, hold the headgear with both hands, position the mask on your nose, and pull the headgear over your head. Make sure that the cup of the mask is centered over your nose, and align the Head Strap to the center of your head.
5. Pull the headgear over the back of your head until the Neck Strap is snug just below your ears.
6. If you experience leaks, please refer to the “Adjusting for Leaks” section.
7. Lie down in your normal sleeping position. If any leaks occur, readjust the mask.
8. Video mask fitting instructions are available on our web site at:

www.circadiance.com/help

Adjusting for Leaks - If you experience leaks, try these steps:

1. With your CPAP or bilevel machine on and at your prescription pressure, pinch the portion of the mask directly over your nose, pull the mask slightly away from your face, towards your feet, and gently release.
2. If you experience leaks from the sides of the mask, adjust the Neck Straps by unhooking the tab from the strap and pulling the straps out to tighten the mask, then re-latching the tab to the strap.
3. If you experience leaks into your eyes, adjust the Head Strap.
4. At times it is possible to stop leaks and get a better fit by loosening the mask just a bit, then repeating Step 1.
5. Leaks can also occur if the headgear straps are too tight or too loose. With practice you should find the snug but comfortable fit that works for you.

Removing the Mask

1. To remove the SleepWeaver, pull the mask away from your nose and lift the headgear off your head.
2. You will not need to loosen the headgear straps. With practice you will be able to pull it on and off after you have adjusted the fit for yourself.

Home Cleaning Instructions - Individual Use

- The mask can be cleaned by hand washing with a mild nonabrasive soap.
- You may also wash the SleepWeaver in a clothes washing machine using mild laundry detergent. (follow detergent instructions) Wash in cold or warm water, never use hot water.
- Line dry. (approximate dry time is from 6 to 12 hours depending on humidity)
- Do not dry the SleepWeaver mask in the clothes dryer.
- After a period of use, the medical tape that attaches the swivel to the mask will lose its adhesion and will need to be replaced.
- The swivel is attached to the mask using latex free white medical tape. You may remove the swivel and clean it separately by soaking it in a solution of 1 part vinegar to 3 parts water (1/3 cup vinegar to 1 cup water).
- Thoroughly rinse the swivel after soaking then let dry.
- Re-attach the swivel with latex free medical tape.
(please refer to the Swivel sections)

Multi-Use Cleaning Instructions - Sleep Labs

- The mask should be disinfected between patients using a high level chemical disinfectant like Control III[®]. (disinfection must be performed in accordance with disinfection agent manufacturer's instructions)
- After disinfection, the mask must be machine washed using mild laundry detergent. (follow detergent directions) Wash in cold or warm water, never use hot water. Line dry, DO NOT dry the SleepWeaver in the clothes dryer. (approximate dry time is from 6 to 12 hours depending on humidity)
- Remove, clean, and replace the swivel between each patient.
(please refer to the Swivel sections)



- The mask must not be disinfected using 2% glutaraldehyde solution, heat sterilization, hot water pasteurization, autoclaving, or ethylene oxide gas sterilization. Do not use bleach or solvents containing trichloroethylene.

Removing the Swivel - *It is not necessary to remove the swivel and replace the tape each time you wash the mask unless you have a chronic condition and your physician or RT has advised you to do so or if the mask is used in a Sleep Lab environment.*

1. Turn the mask inside out to expose the white medical tape attaching the swivel to the mask and remove the old tape.
2. Push the swivel through the nose hole and out of the mask for cleaning.
3. Clean the mask by following the cleaning instructions above.
4. Clean the swivel separately by soaking it in a solution of 1 part vinegar to 3 parts water. Rinse well after soaking then let dry.

Re-attaching the Swivel - Use latex free, waterproof, white medical tape

1. After both the swivel and mask are completely dry turn the mask inside out again.
2. Place the swivel inside the mask with the "male" (ridge) sticking out 1/2 inch from the fabric.
3. Cut a piece of white latex free medical tape approximately 10 1/2 inches long
4. Hold the swivel in the mask with one hand while starting tape next to the ridge. Make sure your tape is 1/2 way over the fabric and 1/2 over the swivel.
5. Wrap the tape around the swivel and mask at least 3 times.
6. Cut off excess tape if necessary and finger smooth away any wrinkles with your finger.

Instructions are also available online at www.circadiance.com/help



Medical Information

The SleepWeaver does not contain latex materials. If you have ANY reaction to any part of this mask or headgear discontinue use and consult your physician or RT.

Exhalation Valve

There is no jet of air with exhalation. Venting is through a series of small pinholes. The flow through the exhalation valve plus leaks from the swivel and other sources is:

Pressure(cmH20)	4	5	10	15	20
Flow (lpm)	12.9	14.7	24.9	33.9	40.0



Cautions & Warnings

- In the USA, federal law restricts this device to sale by, or on the order of, a physician.
- This system should be used only with CPAP or bilevel systems recommended by your physician or respiratory therapist.
- This mask should not be used unless the CPAP or bilevel system is turned on and operating properly.
- The vent holes should never be blocked.
- Please refrain from smoking while using this product
- Your mask must NOT be disinfected using 2% glutaraldehyde solution, heat sterilization, hot water pasteurization, autoclaving, or ethylene oxide gas sterilization. Do not use bleach or solvents containing trichloroethylene.

Limited Warranty

www.circadiance.com/help