
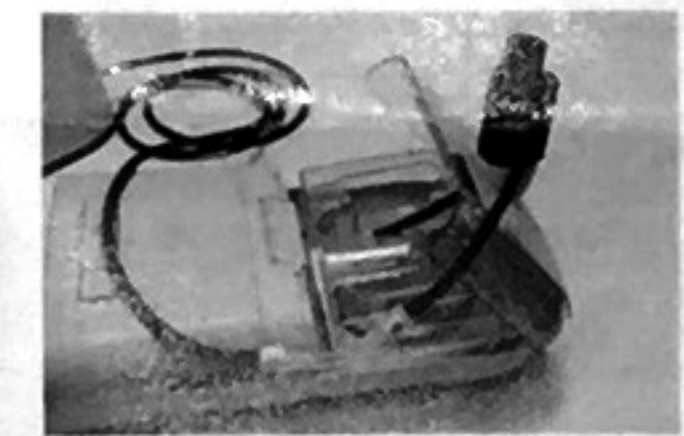
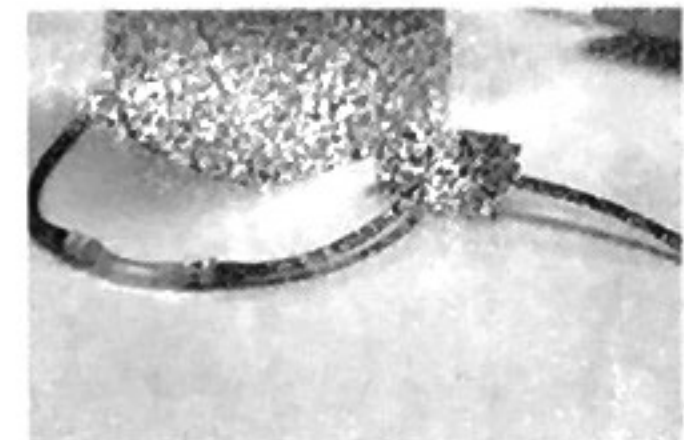


## Installing Your Dream Station Adapter

1. Remove the heated hose from your DreamStation.
2. Remove the SoClean injection fitting along with its short hose from the main SoClean hose.
3. Connect the main SoClean hose to side port of the DreamStation adapter.
4. Open the reservoir lid on your Dream Station.
5. Connect the DreamStation adapter to your DreamStation, feed the injection tubing down into the water reservoir.
6. Close the DreamStation reservoir lid and connect your heated hose to the DreamStation adapter.

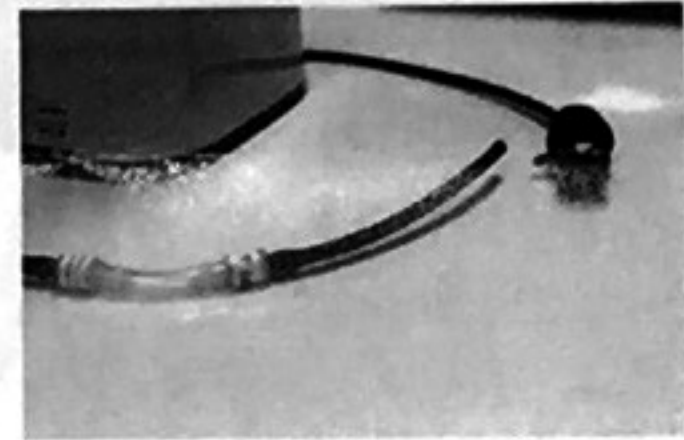

# Using your Philips Respironics System One 60 Series Heated Tubing Heated Humidifier Adapter with the SoClean CPAP Sanitizer


Remove your original humidifier reservoir from your CPAP unit.

Determine which of the two new reservoir lids match your original one and install the new matching lid onto your new humidifier reservoir base. The extra lid can be discarded.

Insert your new adapted humidifier reservoir into your CPAP unit like you normally would.


Disconnect the short length of hose from the end of the check valve assembly on your SoClean (see Figure A). Also attached to this hose will be a black plastic fitting and another short length of hose. You will not be needing these items.

**Important:** *Make sure you leave the check valve assembly connected to the SoClean hose.*

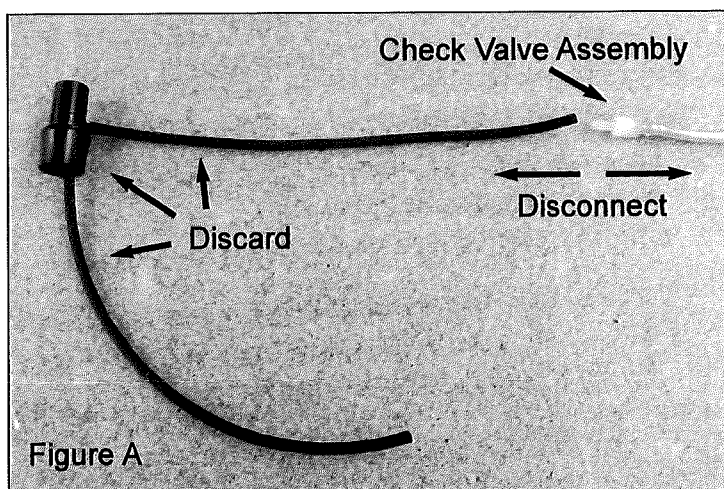

Connect the new hose that is protruding from your adapted humidifier to the check valve on the end of the SoClean hose (see Figure B).

After adding water, your CPAP is now ready for sanitizing or regular usage.

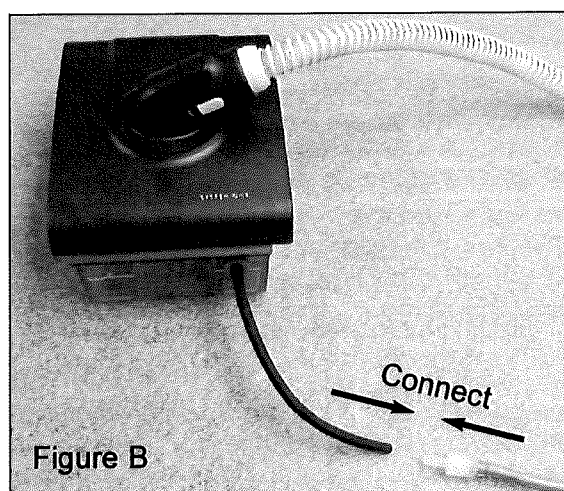


Provided for your reference by

**cpapXchange.com**




## Installing Your S9 Lid Adapter


1. Disconnect the SoClean hose from the fitting by pulling apart at the connection point in the illustration. You will not be using the plastic fitting and its short black hose.

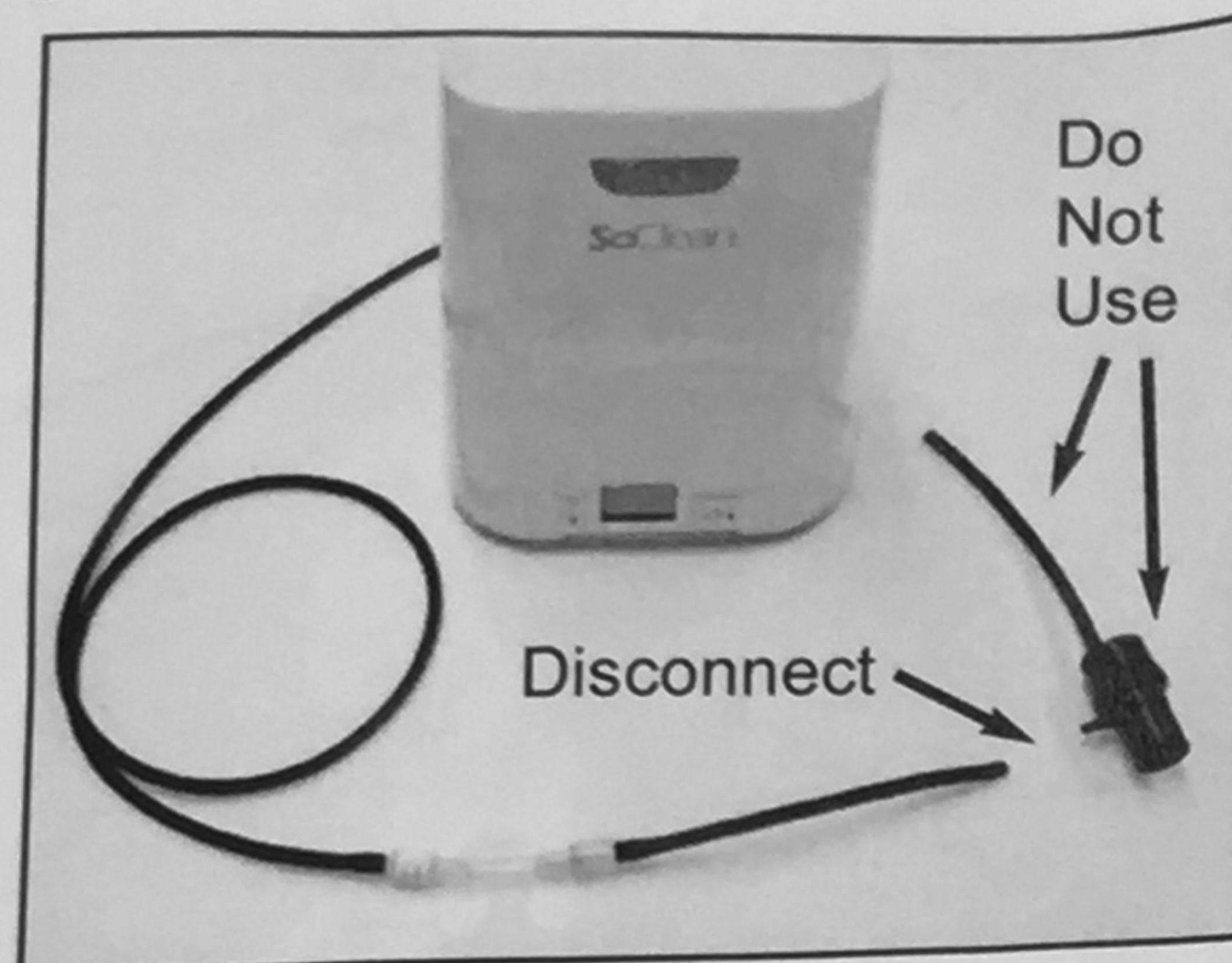


2. Remove the original lid from your humidifier by opening the lid about 3 inches, grasping it near the hinge and pushing firmly away from the unit.

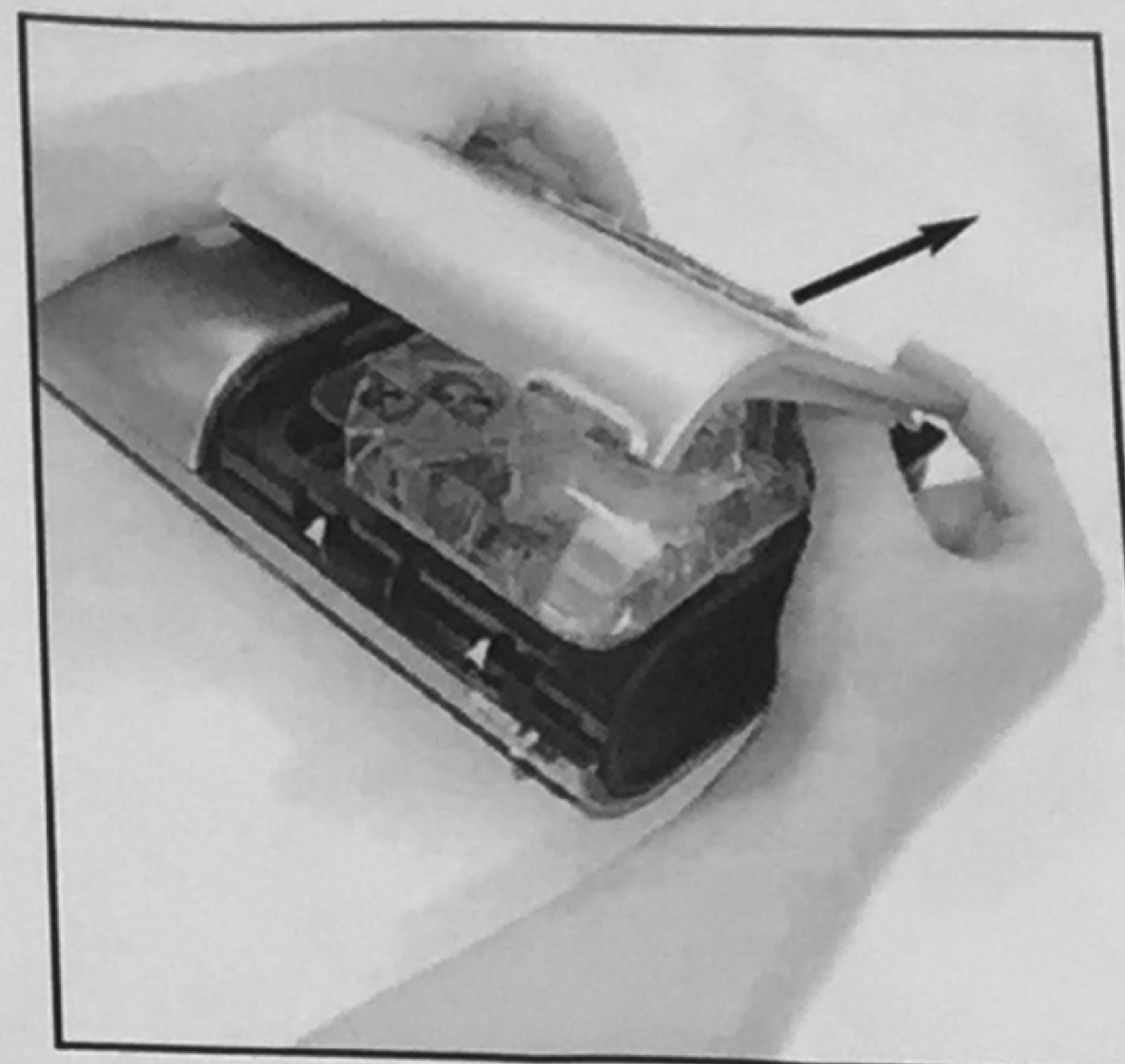


3. Remove the silicone gasket from the original lid and install it onto the new lid, making sure the gasket lies flat and fits over the lid tabs. Install the new lid by following step 2, in reverse.

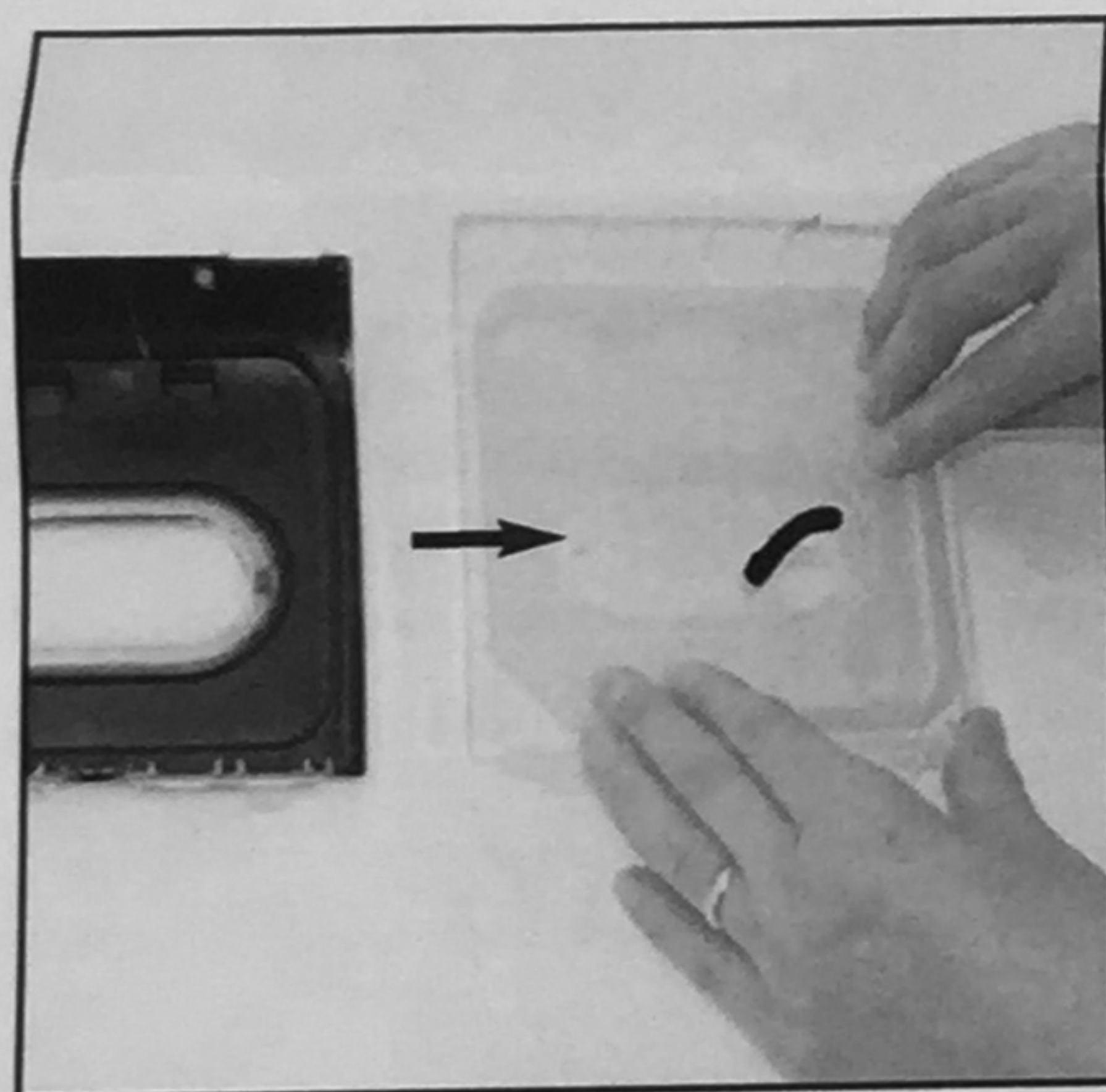


4. When closing the new lid, make sure that the hose on its underside passes into the reservoir opening and rests near the bottom.

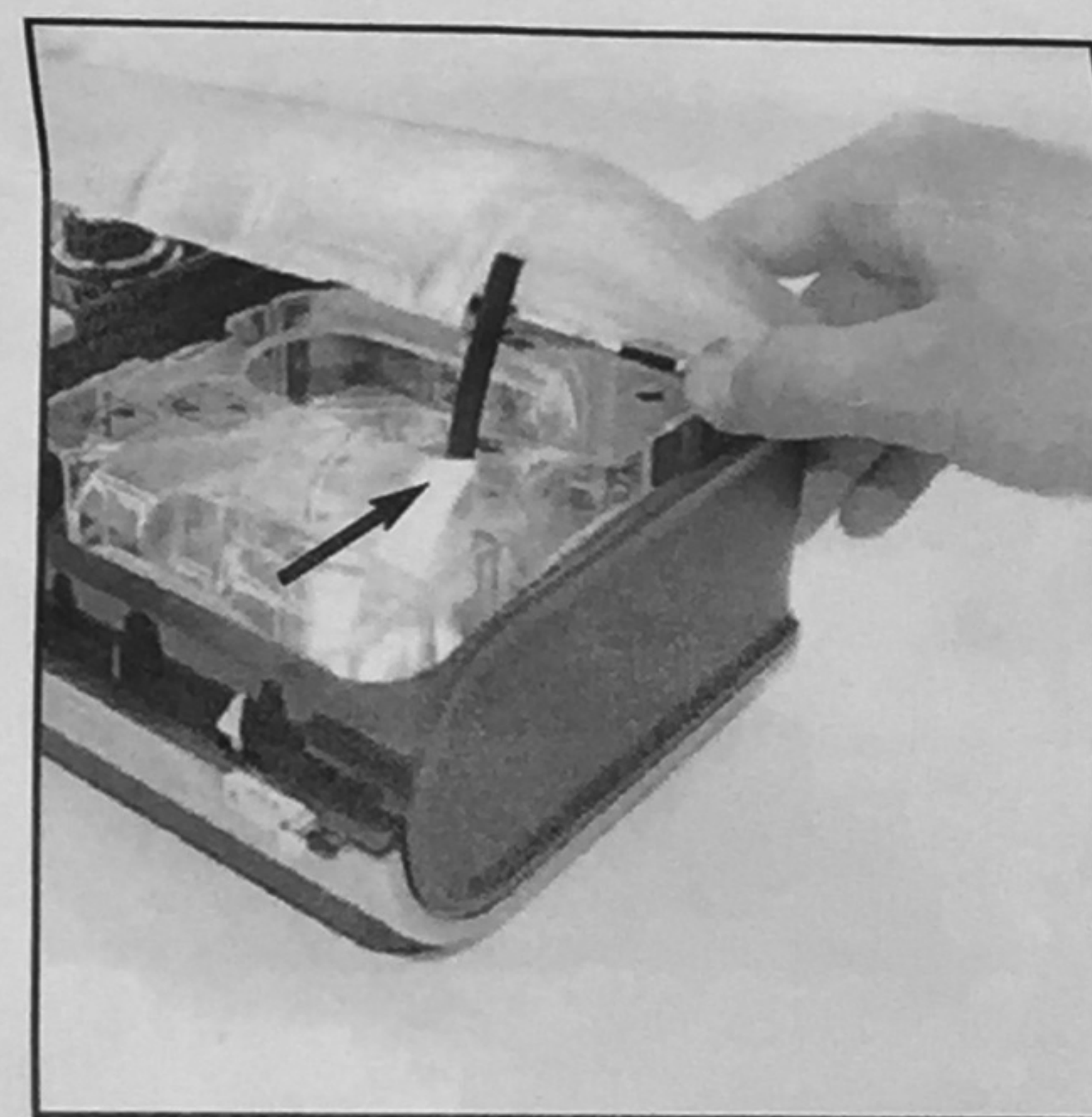

5. Attach the SoClean hose to the top stem of the new lid. Your CPAP is now ready for use.

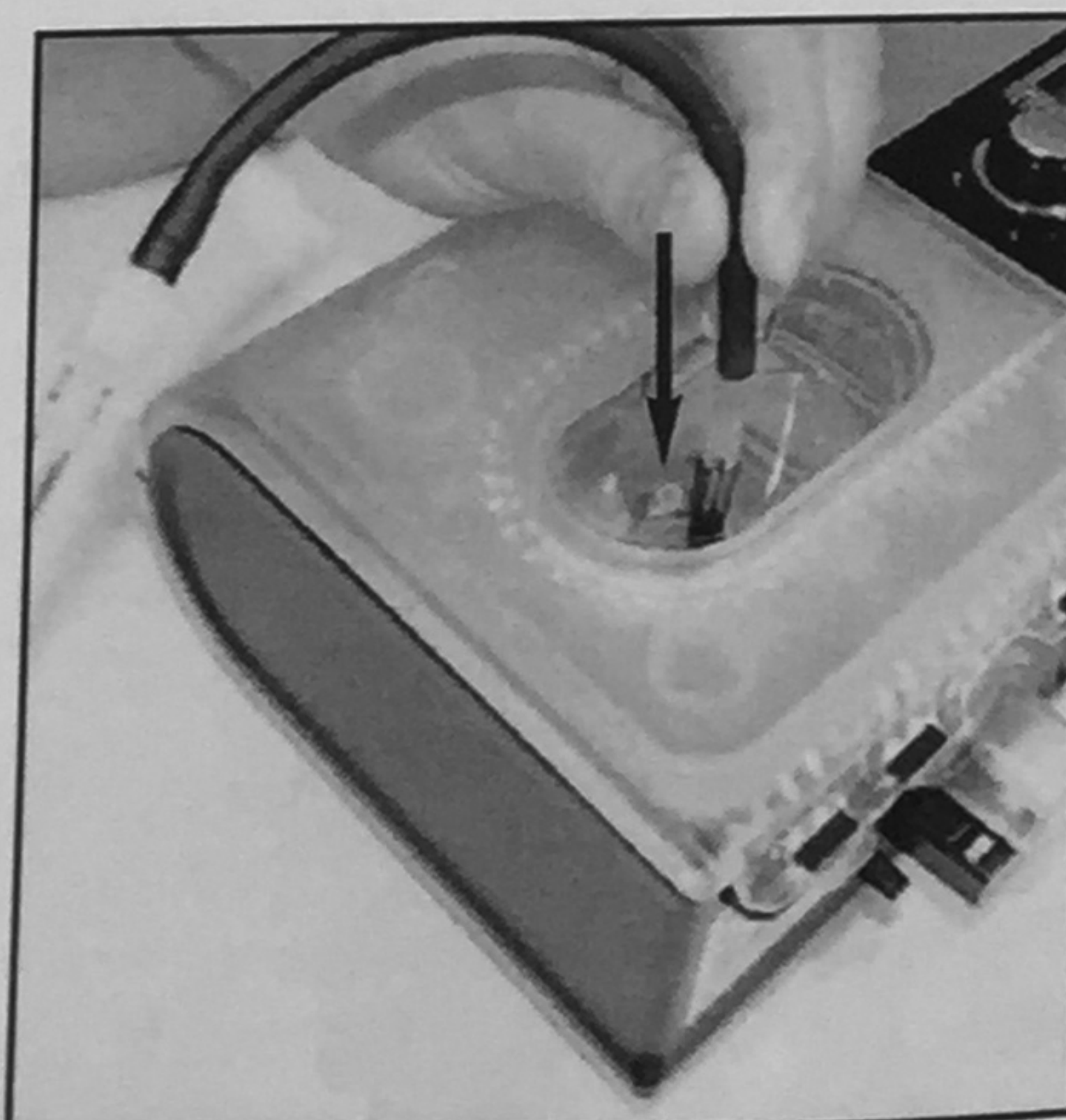


6. To achieve the best seal, firmly push down on the lid top, and slide the latch toward the left.

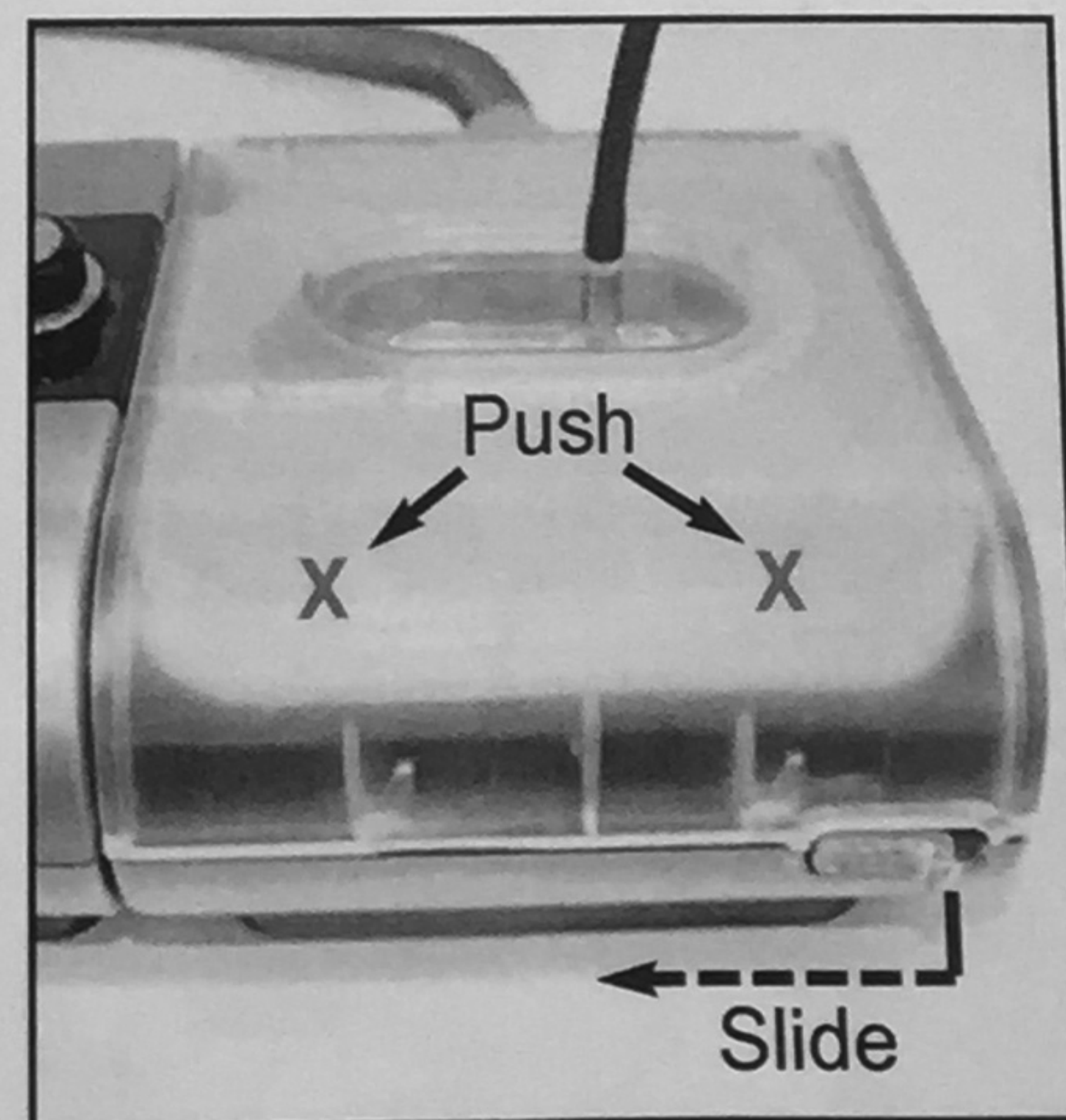





# Installing Your Replacement Humidifier Lid Adapter

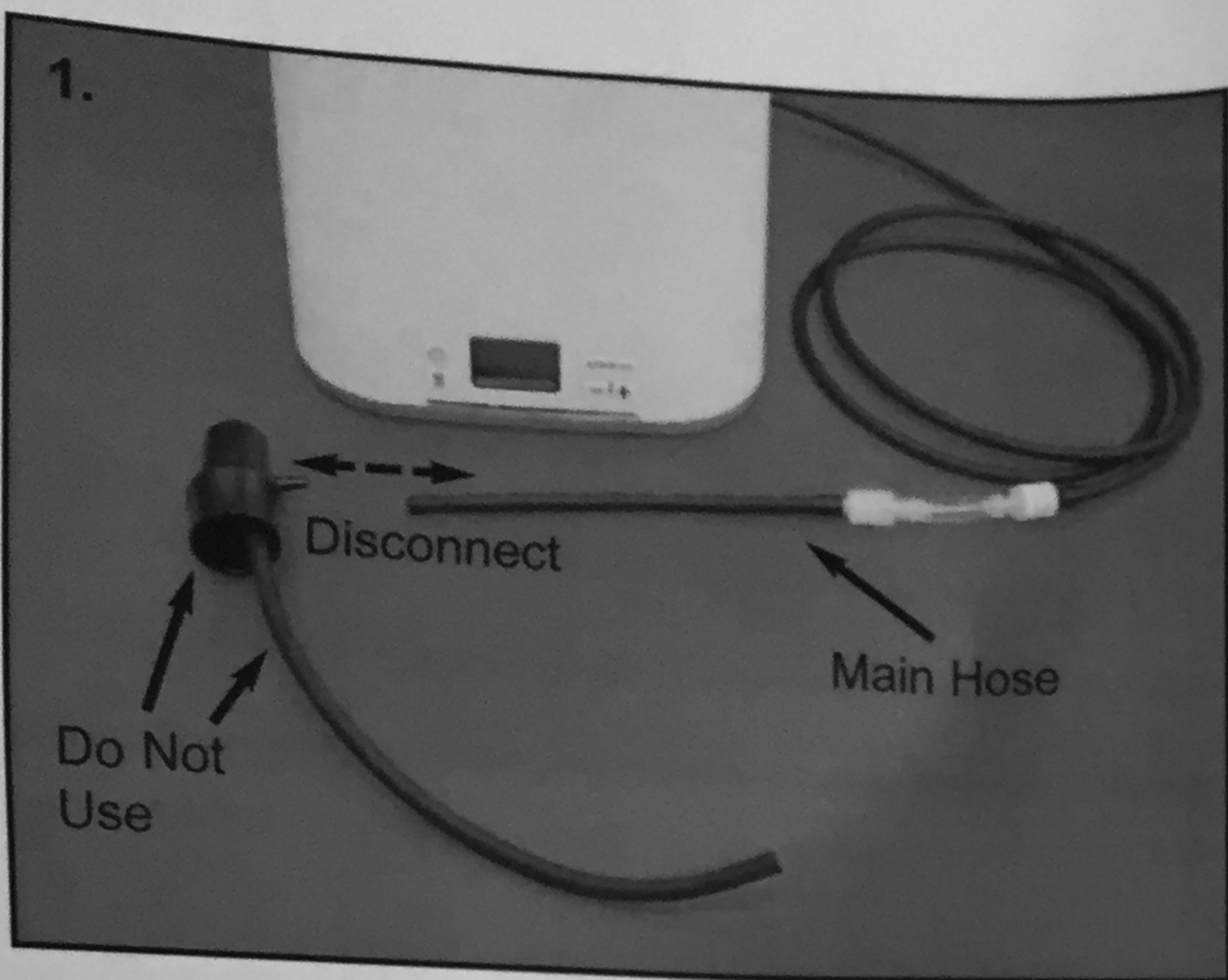

1. Disconnect the SoClean fitting along with its short hose from the main hose. You will no longer need the fitting and attached hose.

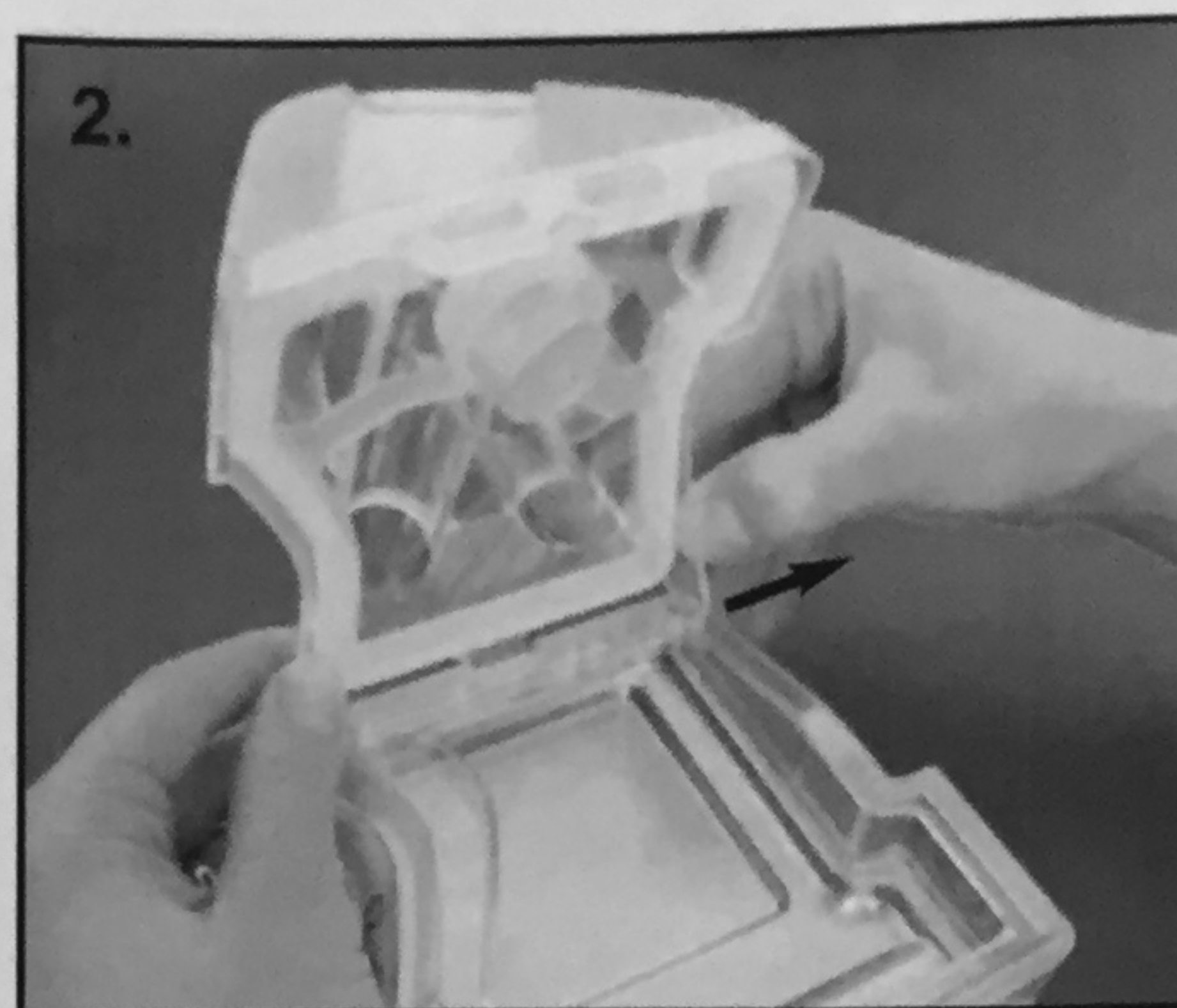


2. Remove original lid from the reservoir base by opening the lid fully, pushing one side of the hinge tab outward and freeing the lid from the base.

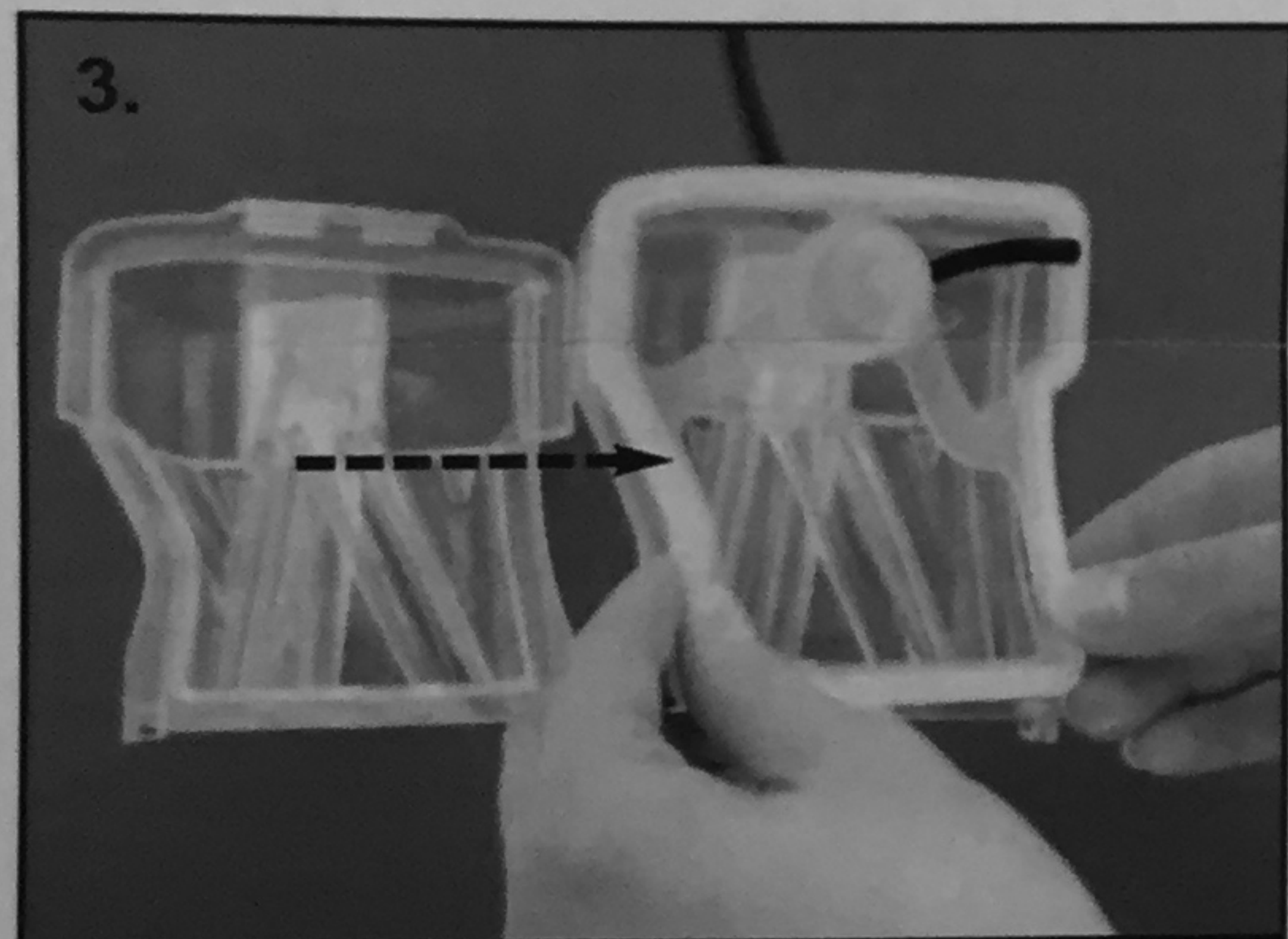

3. Lift gasket from the original lid and transfer it to the new lid. Connect new lid to the reservoir base by following step 2 in reverse.

4. Connect the short reservoir hose with its plastic coupling to the main hose of your SoClean by pushing the plastic coupling into the hose end. Your reservoir is now ready for use.

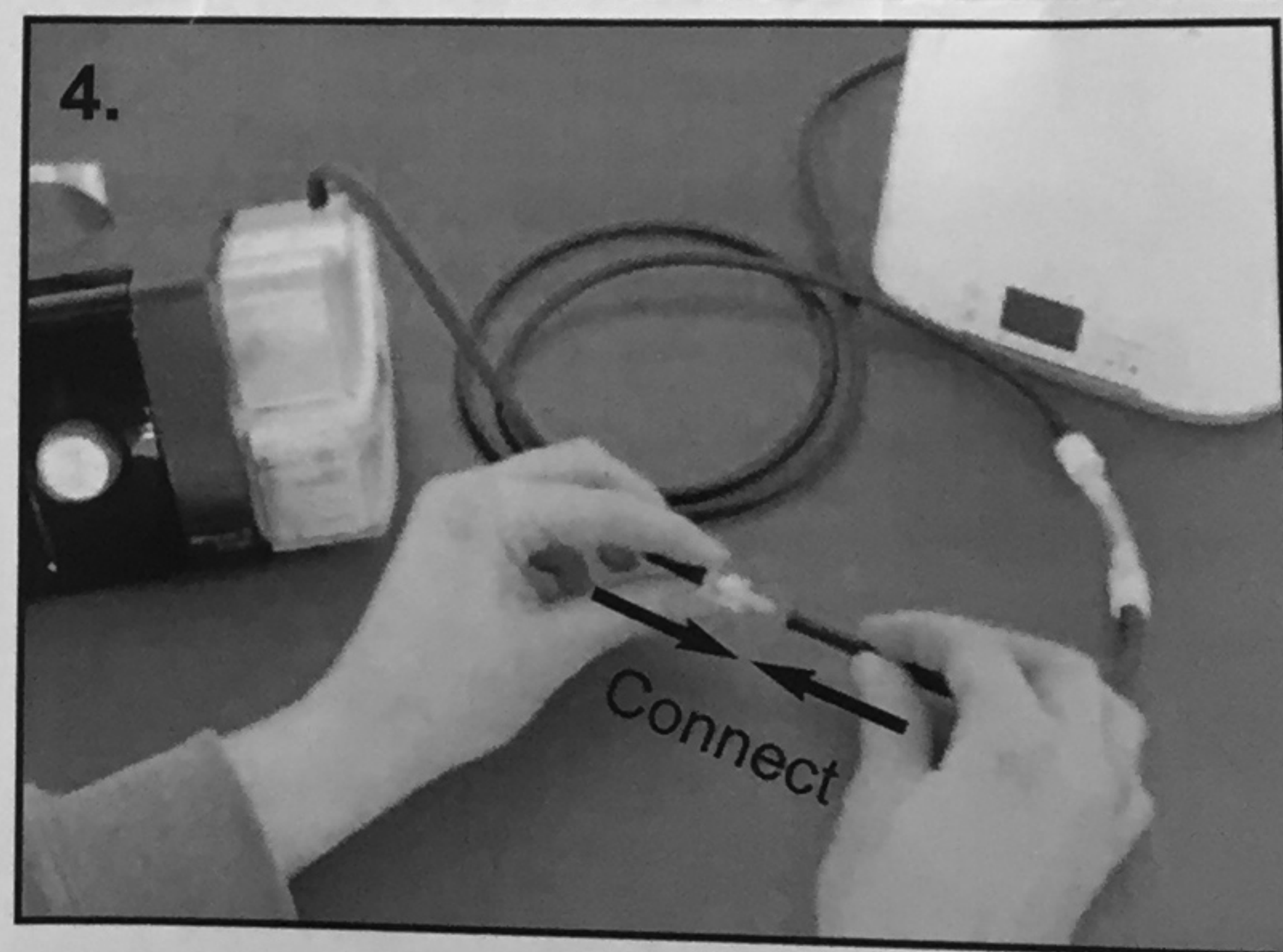


## Easy-Disconnect Coupling

This coupling will allow you to quickly separate and reconnect your reservoir hose, making it easy to remove your reservoir for filling and maintenance.

To use the easy-disconnect coupling, hold both sides of the plastic coupling, unscrew and pull apart. The hose will separate at the coupling. Reverse this action to reconnect the hose.

**IMPORTANT:** You must reconnect your hose before operating your CPAP. If you fail to do this, the water in your humidifier will be pushed out through the hose and flooding will result. Better Rest Solutions is not responsible for damage resulting from any mishandling of its equipment.

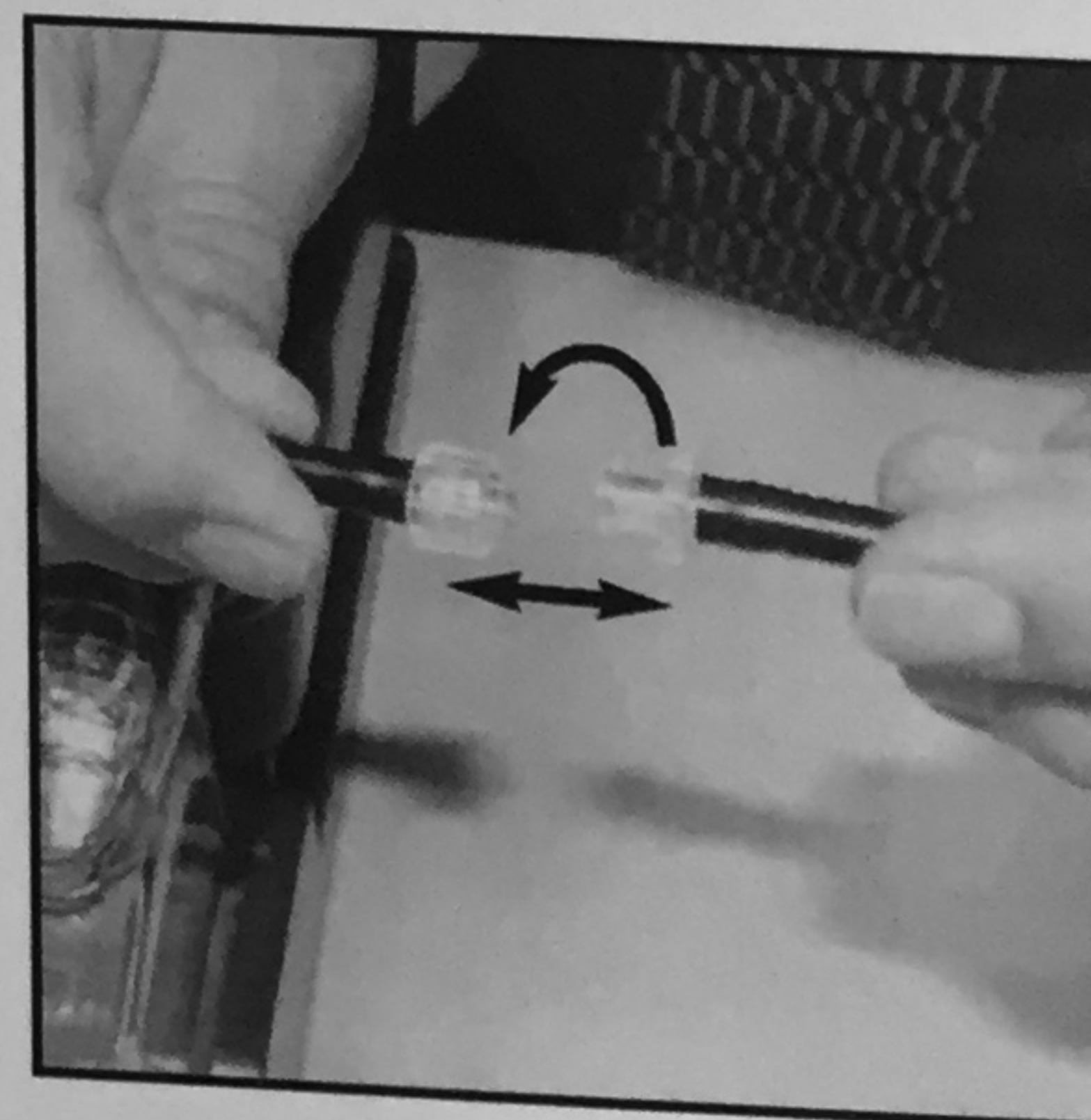



## **SoClean Heated Hose Adapter for use with Fisher & Paykel Icon Humidifier and Heated Hose**

This adapter integrates the SoClean CPAP sanitizer into your Fisher & Paykel CPAP equipment and allows for the use of your heated hose.

Note: The black fitting that comes connected to your SoClean unit will need to be removed and replaced with the gray adapter elbow.

1. Remove the short length of black hose from the black SoClean fitting by holding the fitting in one hand and firmly pulling it out.
2. Attach this length of hose to the *inside* stem of the new elbow adapter. (A small amount of water can be used on the stem to help the hose slide on easier.)
3. Remove the longer length of hose from the top of the black SoClean fitting in the same manner as outlined in Step 1.
4. Attach this longer hose to the *outside* stem of the new elbow adapter.
5. Remove the original elbow fitting from your Fisher & Paykel humidifier and hose.
6. Connect the new elbow adapter to your humidifier the same way you normally would, only now, feed the short black hose into the humidifier reservoir. (Note: it may be necessary to trim off some of this hose so that its end rests on or near the bottom of the reservoir.)
7. Attach your heated hose to the new elbow adapter in the same way you normally would.

Provided for your reference by

**cpapXchange.com**